



YOU CAN DO IT

WITHOUT FINGERSTICKS*

Learn how to remotely share glucose data
with your healthcare provider from your
FreeStyle Libre 2 reader


FreeStyle
Libre 2


life. to the fullest.®
Abbott

*Fingersticks are required if your glucose alarms and readings do not match symptoms or when you see Check Blood Glucose symbol during the first 12 hours.

See last page for Important Safety Information.

©2021 Abbott. ADC-24579 v3.0 09/21

Page 1 of 14

OVERVIEW

What you will need:


- Your FreeStyle Libre 2 reader
- Your USB cable
- Access to a valid email address
- Access to the latest version of your web browser - refer to the Patient LibreView Minimum System Requirements page:
<https://pat.libreview.io/articles/min-requirements>



FreeStyle Libre 2
reader and sensor



FreeStyle Libre 2 reader
USB cord

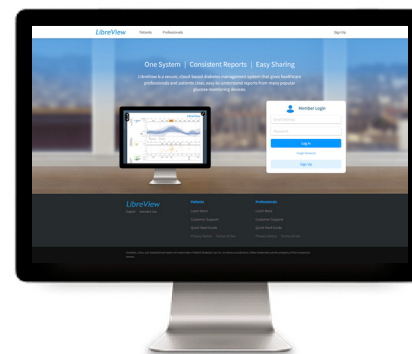

Visit LibreView.com on a web
browser from your computer

Steps to share glucose data:

This guide will walk you through the following 3 steps:

STEP 1: Create a LibreView account online

STEP 2: Link to your doctor's practice

STEP 3: Upload data to share

Note the FreeStyle Libre 2 system is not for use with the FreeStyle LibreLink* app or FreeStyle Libre 14 day reader

If you already have a LibreView[†] account, you can use it to sign-in on your web browser.


*The FreeStyle LibreLink app is only compatible with certain mobile devices and operating systems. Please check our website for more information about device compatibility before using the app. Use of the FreeStyle LibreLink app requires registration with LibreView.

†The LibreView data management software is intended for use by both patients and healthcare professionals to assist people with diabetes and their healthcare professionals in the review, analysis and evaluation of historical glucose meter data to support effective diabetes management. The LibreView software is not intended to provide treatment decisions or to be used as a substitute for professional healthcare advice.


STEP 1: CREATE A LIBREVIEW ACCOUNT

If your healthcare provider sent you a LibreView invitation, follow the steps below. Otherwise, skip to the next page.

1. You will receive an email invitation from LibreView
 - a. Open the email and click on the **Accept Invitation** button

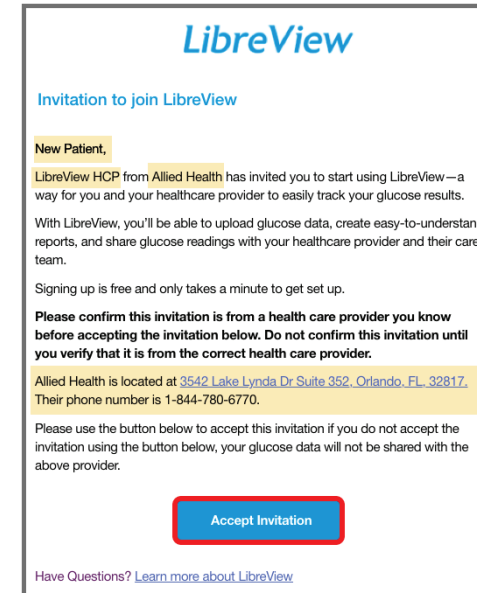

2. Your web browser will open the invitation
 - a. If you are an existing LibreView user, click on the **Log In** button
 - b. If you are new to LibreView, click on the **Get Started** button

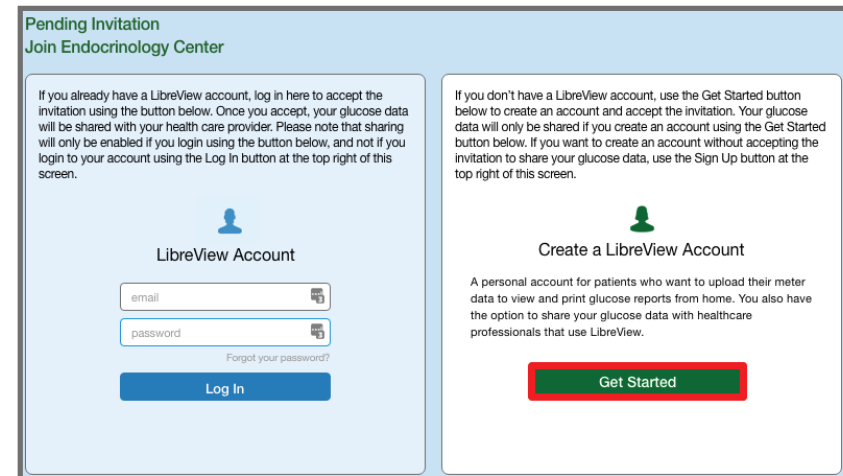


The LibreView data management software is intended for use by both patients and healthcare professionals to assist people with diabetes and their healthcare professionals in the review, analysis and evaluation of historical glucose meter data to support effective diabetes management. The LibreView software is not intended to provide treatment decisions or to be used as a substitute for professional healthcare advice.

STEP 1: CREATE A LIBREVIEW ACCOUNT

If you already have a LibreView account skip to Step 2: Link to Your Doctor's Practice.


If you did not receive an email invitation from your healthcare provider, follow these steps to create an account:

1. Open www.LibreView.com in your web browser
2. Click on **Sign Up** to create a new account

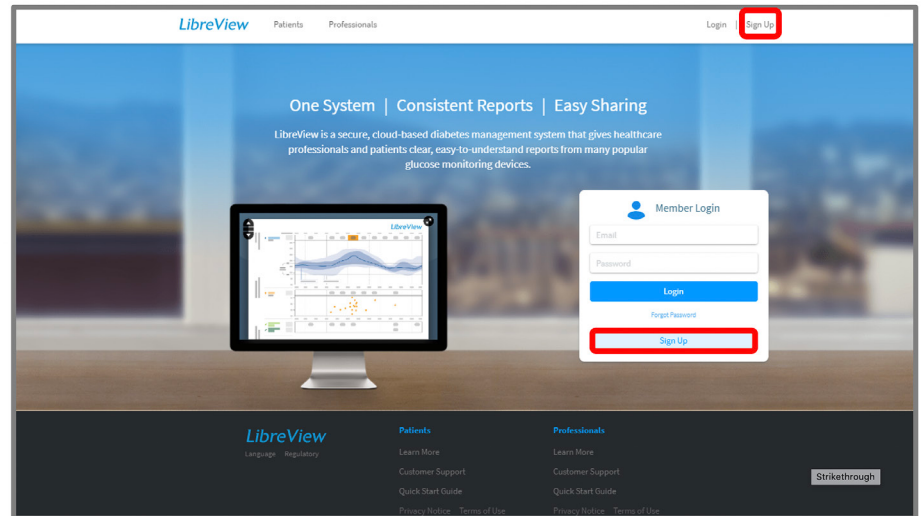

LibreView Account sign-up can be done in minutes:

3. Select **LibreView Account** and click **Continue** to create a patient account

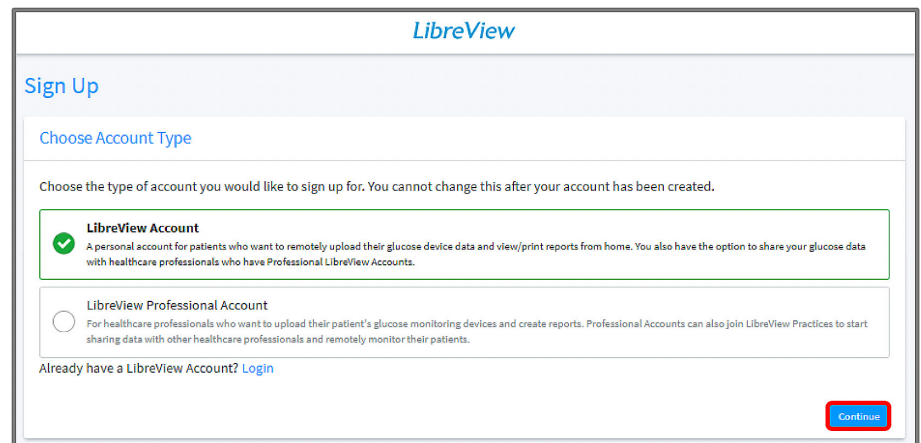

The LibreView data management software is intended for use by both patients and healthcare professionals to assist people with diabetes and their healthcare professionals in the review, analysis and evaluation of historical glucose meter data to support effective diabetes management. The LibreView software is not intended to provide treatment decisions or to be used as a substitute for professional healthcare advice.

STEP 1: CREATE A LIBREVIEW ACCOUNT

Account Sign-up Steps:

1. Select the appropriate **Country / Region of Residence**
2. Accept the **Terms of Use** (step not shown)
3. Accept the **Privacy Notice** (step not shown)

4. Enter Your Account Information

- a. First Name
- b. Last Name
- c. Date of Birth

5. Enter Login Information

- a. Valid Email Address
- b. Create a password and store in a secure place

The screenshot shows the 'Sign Up Patient Account' page at step 1 of 6. The title is 'Country / Region of Residence'. Below the title, there is a text box with the instruction: 'Please choose your country / region of residence. Country / region of residence is used to determine where your data is stored. You cannot change this once you have created your account.' Below this is a dropdown menu currently set to 'United States'. At the bottom left is a 'Back' button and at the bottom right is a 'Next' button.

The screenshot shows the 'Sign Up Patient Account' page at step 4 of 6. The title is 'Account Information'. It contains three input fields: 'First Name', 'Last Name', and 'Date of Birth'. The 'Date of Birth' field is a date picker with 'MM', 'DD', and 'YYYY' components. At the bottom left is a 'Back' button and at the bottom right is a 'Next' button.



The screenshot shows the 'Sign Up Patient Account' page at step 5 of 6. The title is 'Login Information'. It contains an 'Email Address' field with the example 'example@email.com', a 'Password' field, and a 'Re-enter Password' field. To the right of the password fields are 'Password Requirements' listed as: '8 to 36 characters', 'At least 1 number', and 'At least 1 special character !@#\$%^&*()+=~_-'. Below these fields is a checkbox labeled 'Please send me information about Abbott Diabetes Care and other Abbott products and services. (You can unsubscribe at any time.)'. At the bottom left is a 'Back' button and at the bottom right is a 'Next' button.

The LibreView data management software is intended for use by both patients and healthcare professionals to assist people with diabetes and their healthcare professionals in the review, analysis and evaluation of historical glucose meter data to support effective diabetes management. The LibreView software is not intended to provide treatment decisions or to be used as a substitute for professional healthcare advice.

STEP 1: CREATE A LIBREVIEW ACCOUNT

Account Sign-up Steps (continued):

1. You will receive an email to verify your account from LibreView. The email is sent to the email address that you provided in the previous step
 - a. Open the email and click on the **Verify Email** button
 - b. Switch back to your web browser and click **Next**

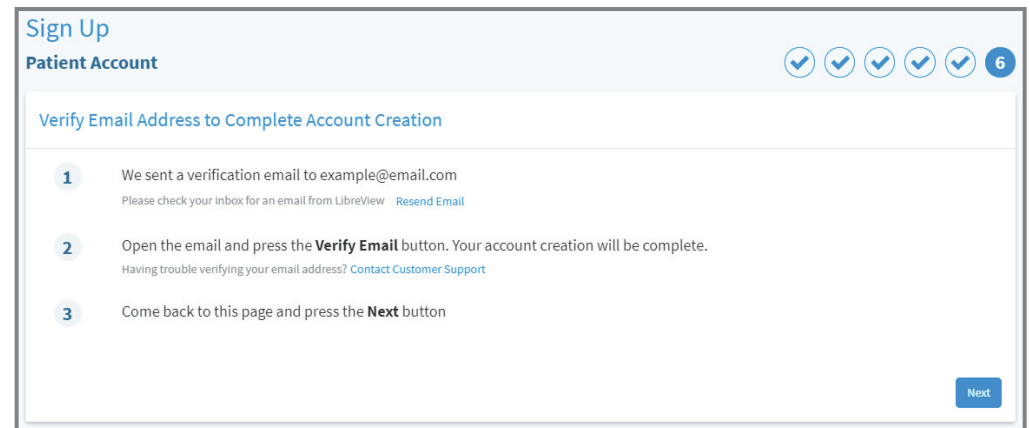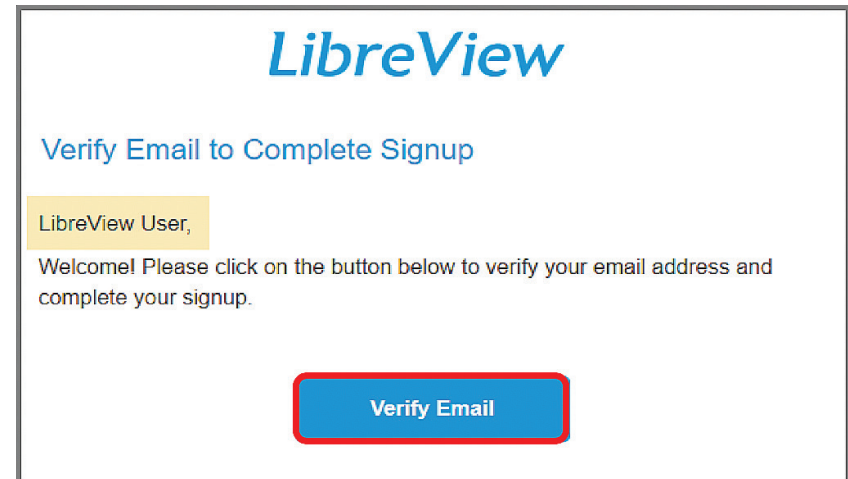


The LibreView data management software is intended for use by both patients and healthcare professionals to assist people with diabetes and their healthcare professionals in the review, analysis and evaluation of historical glucose meter data to support effective diabetes management. The LibreView software is not intended to provide treatment decisions or to be used as a substitute for professional healthcare advice.

STEP 1: CREATE A LIBREVIEW ACCOUNT

Your LibreView account setup is now complete:

- You may explore tutorials for Getting Started, Upload Devices, or Link a Practice from this page

1. Click **Done** to launch your LibreView account

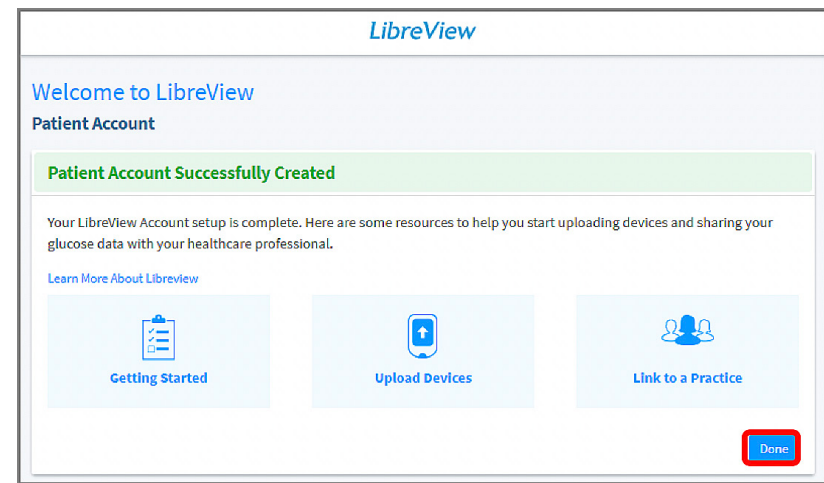

The LibreView data management software is intended for use by both patients and healthcare professionals to assist people with diabetes and their healthcare professionals in the review, analysis and evaluation of historical glucose meter data to support effective diabetes management. The LibreView software is not intended to provide treatment decisions or to be used as a substitute for professional healthcare advice.


STEP 2: LINK TO YOUR DOCTOR'S PRACTICE

If you used an email invitation from your healthcare provider to create or log into your LibreView account then your account is already linked to your healthcare provider and you can skip to the next page. If not, please follow the steps below.

If you would like to initiate a connection with your HCP:


Click on  icon in the upper right corner

1. Select **Account Settings** from the menu

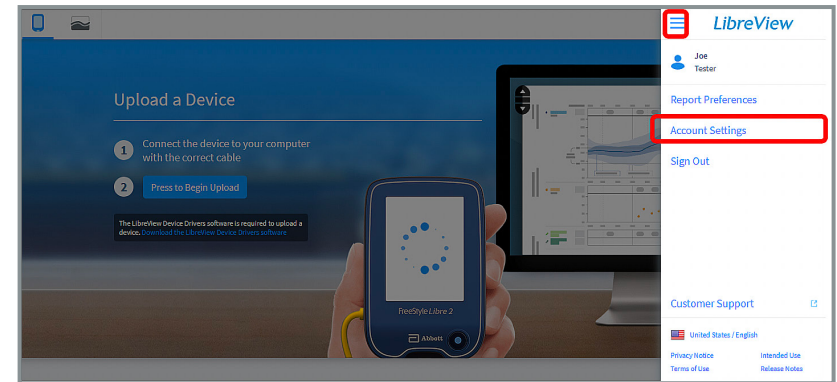

2. Select **My Practices** option
3. Enter the Practice ID provided by your healthcare provider and click **Add**

Note: if you do not have this information, call your provider to request it.

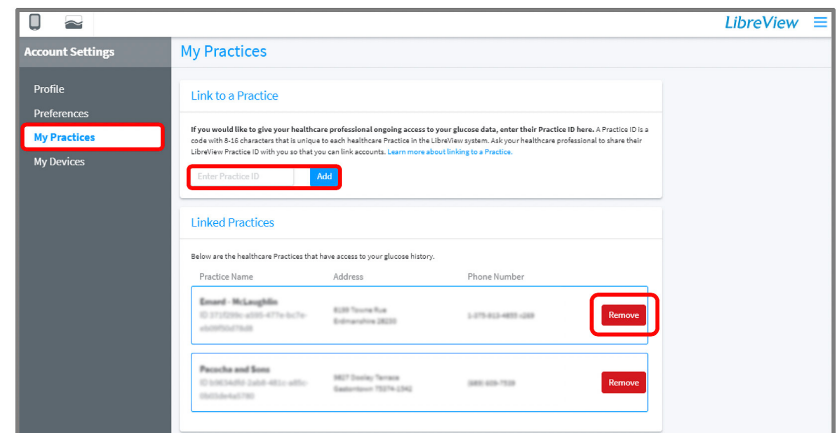


You can also remove your data sharing with a practice by clicking the **Remove** button


The LibreView data management software is intended for use by both patients and healthcare professionals to assist people with diabetes and their healthcare professionals in the review, analysis and evaluation of historical glucose meter data to support effective diabetes management. The LibreView software is not intended to provide treatment decisions or to be used as a substitute for professional healthcare advice.

STEP 3: DOWNLOAD AND INSTALL THE LIBREVIEW DEVICE DRIVERS

1. Click the Download the LibreView Device Drivers software button on the left side of the screen

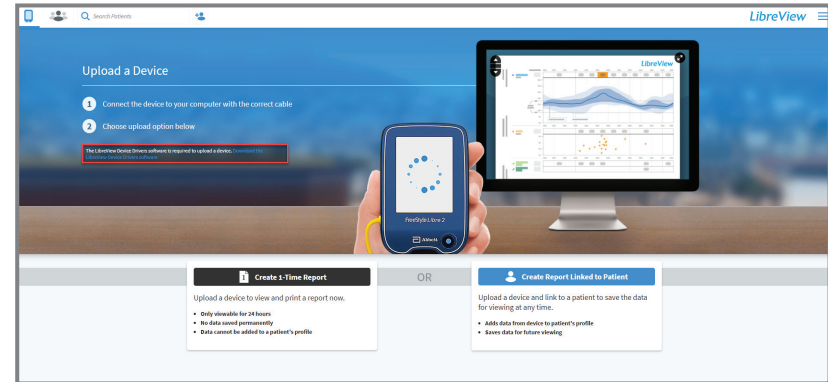

2. When prompted click the Download button specific to your operating system

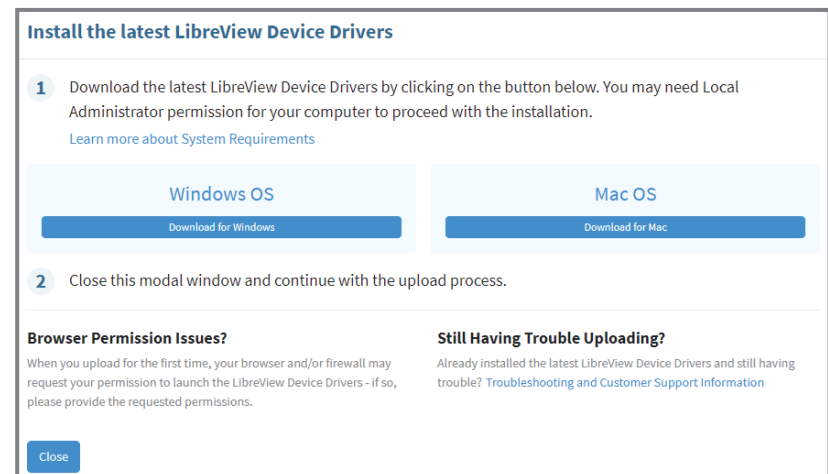


The LibreView data management software is intended for use by both patients and healthcare professionals to assist people with diabetes and their healthcare professionals in the review, analysis and evaluation of historical glucose meter data to support effective diabetes management. The LibreView software is not intended to provide treatment decisions or to be used as a substitute for professional healthcare advice.

STEP 3: DOWNLOAD AND INSTALL THE LIBREVIEW DEVICE DRIVERS (Google Chrome)

If you use Google Chrome, follow the steps below. Otherwise, skip to the next page.


From **Google Chrome**:

- The file will download automatically and will appear in the bottom left corner of your Google Chrome browser
- Click to run the program

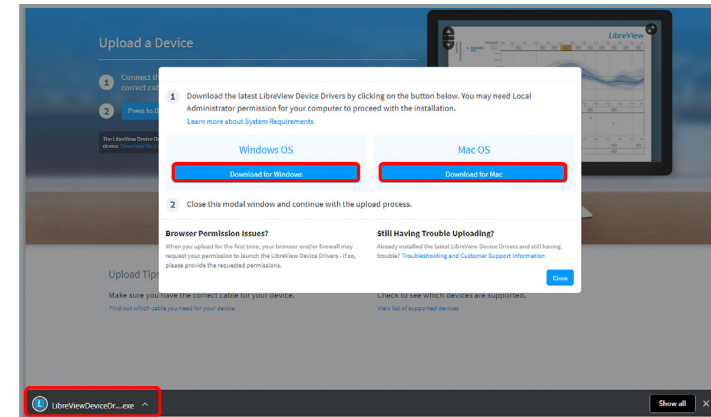

After downloading the file, follow the prompts to install the Device Drivers:

- When the InstallShield Wizard launches, it will prompt you to select the **Preferred language** for installation
- The InstallShield Wizard will then prepare for the installation
- Follow the prompts by clicking **Next** and then **Install**

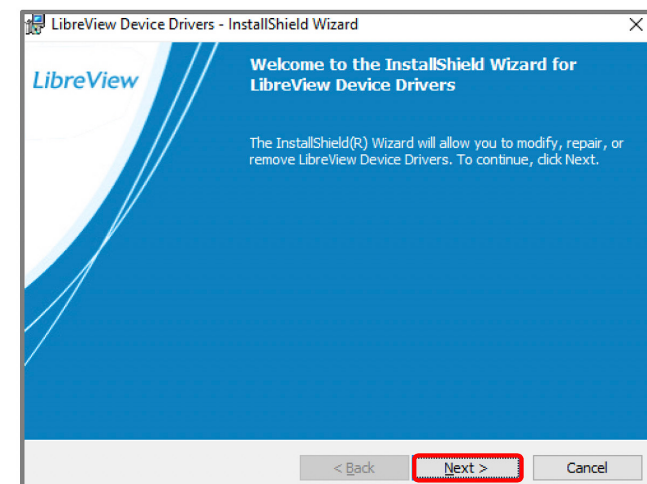

The LibreView data management software is intended for use by both patients and healthcare professionals to assist people with diabetes and their healthcare professionals in the review, analysis and evaluation of historical glucose meter data to support effective diabetes management. The LibreView software is not intended to provide treatment decisions or to be used as a substitute for professional healthcare advice.

STEP 3: DOWNLOAD AND INSTALL THE LIBREVIEW DEVICE DRIVERS (Mozilla Firefox)




If you use Mozilla Firefox, follow the steps below. Otherwise, skip to the next page.

From **Mozilla Firefox**:

- In the pop up download box, click Save File
- The download will appear in the Firefox toolbar, click on the ↓ to locate the downloaded file
- Click on the file name to run

After downloading the files, follow the prompts to install the Device Drivers:

- When the InstallShield Wizard launches, it will prompt you to select the **Preferred language** for installation
- The InstallShield Wizard will then prepare for the installation
- Follow the prompts by clicking **Next** and then **Install**

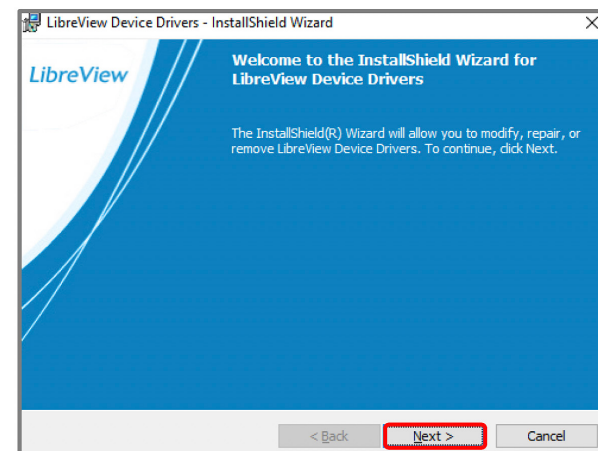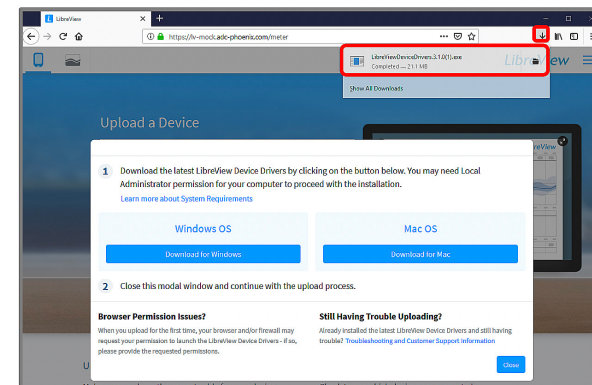
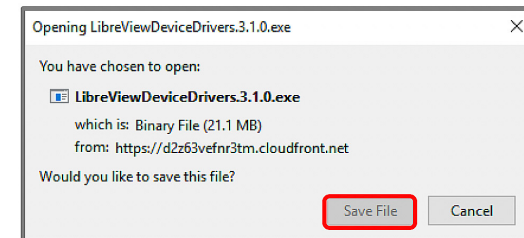


The LibreView data management software is intended for use by both patients and healthcare professionals to assist people with diabetes and their healthcare professionals in the review, analysis and evaluation of historical glucose meter data to support effective diabetes management. The LibreView software is not intended to provide treatment decisions or to be used as a substitute for professional healthcare advice.

STEP 3: DOWNLOAD AND INSTALL THE LIBREVIEW DEVICE DRIVERS (Safari)





From **Safari**:

- The download will appear in the toolbar at the lower right corner of the screen
- Click on the ↓ to locate the downloaded file
- Click on the File name to run

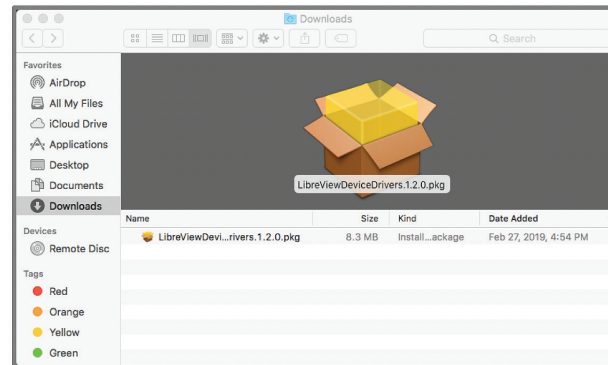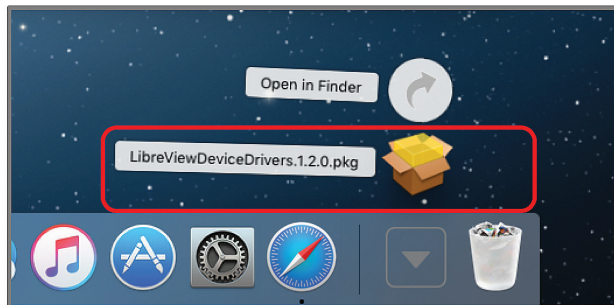

Downloading LibreView Device Drivers – InstallShield Wizard For Mac:

- When the InstallShield Wizard launches, the Wizard will prepare for the installation
- Follow the prompts by clicking **Continue**, **Continue**, **Install** and then **Close**

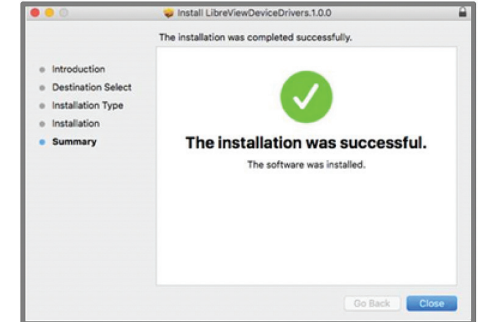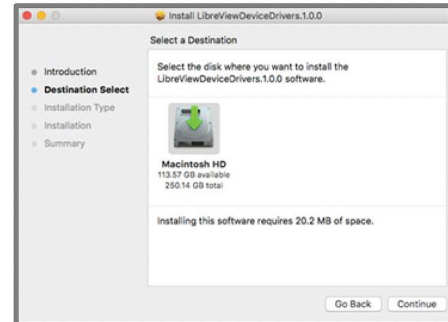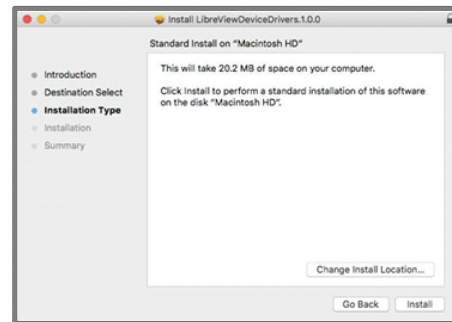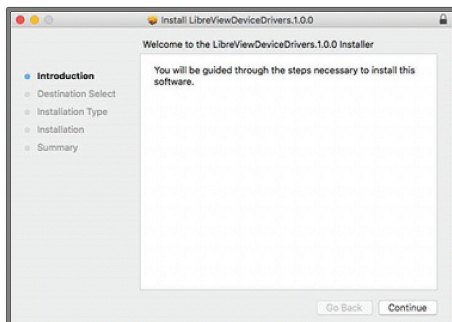


The LibreView data management software is intended for use by both patients and healthcare professionals to assist people with diabetes and their healthcare professionals in the review, analysis and evaluation of historical glucose meter data to support effective diabetes management. The LibreView software is not intended to provide treatment decisions or to be used as a substitute for professional healthcare advice.

STEP 4: UPLOAD DATA TO SHARE

Confirm the Installation is successful:


1. Once the installation is complete, click on the **Press to Begin Upload** button
2. LibreView will automatically detect the LibreView Device Driver is installed and will search for a compatible device

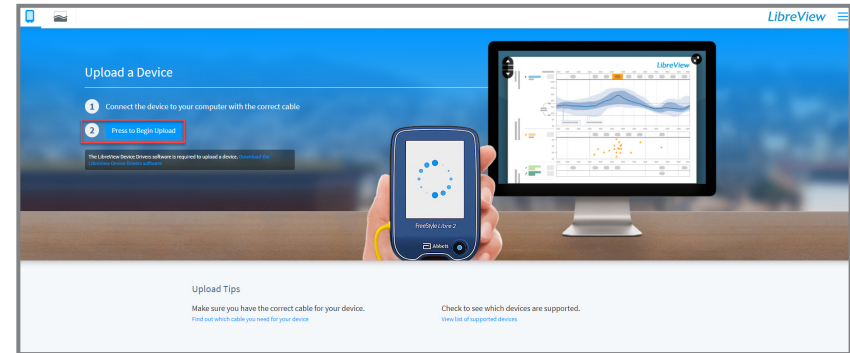

3. Once the device has been detected, the screen will change to **Gathering Device Information** and will launch to My Glucose History so Reports can be retrieved.


Note: Your browser may prompt you with a protocol handler to allow the page to open the “LibreView Device Drivers”. When prompted, click Allow/Open.

4. Your glucose data should now be available for your healthcare provider

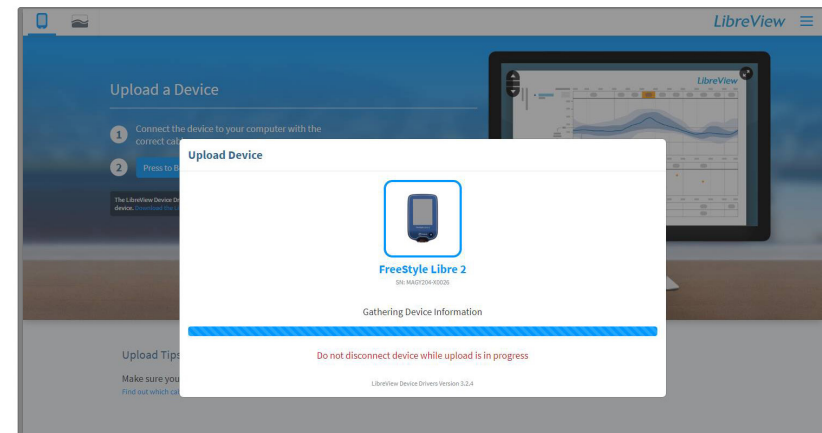


The LibreView data management software is intended for use by both patients and healthcare professionals to assist people with diabetes and their healthcare professionals in the review, analysis and evaluation of historical glucose meter data to support effective diabetes management. The LibreView software is not intended to provide treatment decisions or to be used as a substitute for professional healthcare advice.

If you are a patient and experience problems with installation, please reach out to:

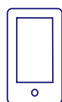

Abbott Customer Care Team 1-855-632-8658

Available 7 days a week, 8AM to 8PM EST, excluding holidays

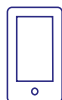

LibreView Tech Support Team 1-855-786-4263



Images are for illustrative purposes only.

The LibreView data management software is intended for use by both patients and healthcare professionals to assist people with diabetes and their healthcare professionals in the review, analysis and evaluation of historical glucose meter data to support effective diabetes management. The LibreView software is not intended to provide treatment decisions or to be used as a substitute for professional healthcare advice.

Important Safety Information

Failure to use FreeStyle Libre 2 system as instructed in labeling may result in missing a severe low or high glucose event and/or making a treatment decision, resulting in injury. If glucose alarms and readings do not match symptoms or expectations, use a fingerstick value from a blood glucose meter for treatment decisions. Seek medical attention when appropriate or contact Abbott at 855-632-8658 or <https://www.FreeStyle.abbott/us-en/safety-information.html> for safety info.

The circular shape of the sensor housing, FreeStyle, Libre, and related brand marks are marks of Abbott. Other trademarks are the property of their respective owners. ©2021 Abbott. ADC-24579 v3.0 09/21 Page 14 of 14