

Get your patients started.



1 How to prep the application site

WASH: Use **non-moisturizing, fragrance-free** soap.

CLEAN: Use an **alcohol wipe** to remove oily residue.

DRY: Allow skin to dry fully before applying the sensor.

2 Apply the sensor

Unscrew cap from applicator and set cap aside. To apply the sensor, press firmly and listen for the click. Pull back slowly after a few seconds.

3 How to get glucose readings

STREAM: Open the **Libre app*** or **reader†** to see a new glucose reading sent automatically to your device.



The FreeStyle Libre 3 system includes the FreeStyle Libre 3 Plus sensor, FreeStyle Libre 3 sensor, FreeStyle Libre 3 app, Libre App, and the FreeStyle Libre 3 reader.

*The FreeStyle Libre systems apps are only compatible with certain mobile devices and operating systems. Please check the Support section of our website for more information about device compatibility before using the apps. Use of the FreeStyle Libre systems apps may require registration with LibreView. †The FreeStyle Libre systems apps and the FreeStyle Libre 3 reader have similar but not identical features.


Fingersticks are required for treatment decisions when you see Check Blood Glucose symbol and when your glucose alarms and readings from the system do not match symptoms or expectations.

See page 2 for Important Safety Information.



4 Understand why BGM and CGM readings may differ

Just as the engine of a train leads the caboose, glucose enters the bloodstream first. Sensor glucose readings eventually catch up, just like the back of a train following the front.

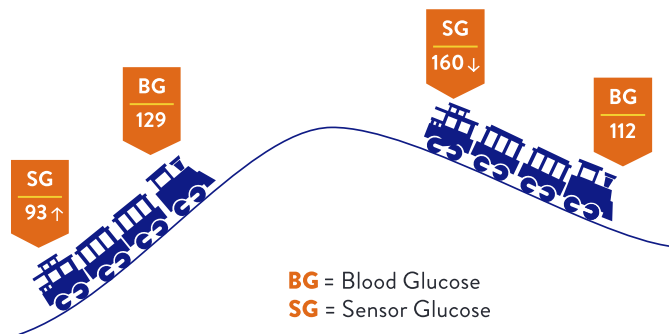

5 Get free, valuable education and support

Join **MyFreeStyle** and get customized resources, such as emails, texts, and videos to help gain insights and manage your diabetes with more confidence¹.



LEARN how food and physical activity affect glucose



MAKE more informed treatment decisions



TRACK your overnight lows



SHARE your glucose data^{†‡}

Enroll today at StartMyFreeStyle.com or scan the QR code to get started.



[Unique variations of the QR code will lead to ADC-42329]



For additional support, call the Abbott Customer Care Team at **1-855-632-8658**, 7 days a week, from 8AM to 8PM Eastern Time, excluding holidays.

Learn More at FreeStyleLibre.us

[†]The FreeStyle Libre systems apps are designed to facilitate data sharing between patients and their healthcare providers and caregivers. [‡]The user's device must have internet connectivity for glucose data to automatically upload to LibreView. **1.** Data on File. Abbott Diabetes Care.

Important Safety Information

Failure to use FreeStyle Libre 3 system as instructed in labeling may result in missing a severe low or high glucose event and/or making a treatment decision, resulting in injury. If glucose alarms and readings do not match symptoms or expectations, use a fingerstick value from a blood glucose meter for treatment decisions. Seek medical attention when appropriate or contact Abbott at 855-632-8658 or FreeStyleLibre.us for safety info.

The sensor housing, FreeStyle, Libre, and related brand marks are marks of Abbott. Other trademarks are the property of their respective owners.

© 2025 Abbott. ADC-74759 v4.0

